eVito COLLECT

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Content:

Important information: please read first!

14

I. Introduction

15

II. eVito collect

15

Card slots

16

Inserting SIM-card

16

Inserting SD-card

17

Inserting the battery

17

Battery status check

18

USB protective cover

18

III. Setup and operation functions

19

Device switch on/off

19

Automatic data transmission

20

Locking your device

21

Energy saving mode

21

Declaration of conformity

22

WEEE notice

22

RoHS compliance

23

Technical specifications

23

Warranty

23

*Please read this user manual thoroughly
before first using this device!*

Important Information Please Read First!

Safety Instructions

- Be sure to keep equipment and accessories away from small children because small parts could be swallowed. Ingestion may cause choking or serious injuries!
- Ensure a minimum distance of 15 cm to be maintained between the device and a cardiac pacemaker. Failure to comply can lead to serious disturbances of the cardiac pacemaker (or other medical devices)! Switch off your device when in doubt - do better to prevent this!
- The use of this device is forbidden in most hospitals, as they affect medical device electromagnetic interference and/or may interfere. Keep in mind the local regulations and turn your device off if necessary!
- If you use another medical device, check with the manufacturer if the unit has a protection against external RF energy. Use this device only if you can exclude any interference with other medical devices.
- Proceed with care and common sense to use the equipment.
- Do not disassemble the device independently. Never insert objects into the slots or other openings of the device.
- Avoid banging or dropping the unit. Treat this electronic product always with care to avoid damage to the device.
- Do not paint your device. The paints can damage the case, stick moving parts and thus damage the unit such as.

I. Introduction

Thank you for choosing **eVito collect**. This data collector in USB stick format will make it possible in a simple manner to receive, store and wirelessly transmit health data, obtained with ANT-enabled instruments. The **eVito collect** can be worn as a key ring and itself requires no active intervention of the operator. Data reception and data transmission are performed fully automatically.

In this manual you learn the basic functions in dealing with your **eVito collect**.

II. eVito collect

Front side

Card slots

The **eVito collect** has 2 card slots. Thus the received data can be stored on the device on the SD-Memory Card. The SD-card is delivered with the device.

The data received via ANT are sent via mobile data connection with the aid of a SIM card. To use the **eVito collect** as mobile data collection unit, you need a SIM card.

The data can also be transferred manually to a computer. In this case, you have to plug the **eVito collect** such as a USB stick into the appropriate slot on your PC or laptop.

Warning!

eVito collect is only receiving data, as long as it is not plugged into a USB slot. The data reception is then only possible if the device is pulled out.

What is ANT?

ANT is a wireless measured value transmission standard. To use the ANT standard, the transfer device and receiving device must be equipped with a suitable module.

Inserting SIM card*

- Remove the cover on the back of the device by dragging it in the direction of the key ring.
- The larger of the two memory card slots is provided for the SIM card. To insert a SIM card, slide the holder collect gently upwards towards the cap or the USB connector.
- Flip the holder upwards.
- Now insert the SIM card with the label facing up. The rounded corner points to the right.
- Lower the holder so that the SIM card of which is covered and slide it down toward the key ring. The holder engages with an audible click.

Inserting SD-card

- The smaller of the two memory card slots is provided for the SD-card. To use an SD-card, slide the holder gently to the right toward the LOCK button on the side of the device.
- Now insert the SD-card with the label facing up. An arrow on the map shows you the direction in which they are inserted (open/close)

** The use of **eVito collect** with SIM card is optional and only necessary, if you want to use the device as a mobile data collection unit. Please have in mind the mobile data connection charges from your mobile provider associated with using a SIM card and the associated transmission and reception of data.*

Inserting the battery

Finally you must insert a battery into the **eVito collect**. Insert the battery so the contacts are located in the upper right corner. The battery is placed over SD and SIM card. Finally, slide the cover on the back, again in the direction of the USB plug. Your **eVito collect** is now operational.

Battery status check

The battery of the **eVito collect** is charged via the USB interface. You can check the battery status at any time during operation on the device itself. Please press the button on the front panel 2 times briefly. The battery status is displayed for 3 seconds with the LEDs that are located above the button.

The LEDs are located inside the unit under the transparent cover. They shine through the white space throughout. The green LED can sometimes appear to be white.

All 3 LEDs blink when the device is fully charged.

The blue and white LED flash when the battery is half full.

The blue LED flash when the battery is low.

If the battery level is too weak, all 3 LEDs flash rapidly for a moment.

If the battery status in the operating mode is too weak, all 3 LEDs flash briefly every minute. This serves as a safety warning. You should charge the device as soon as possible.

USB protection cap

You can use the **eVito collect** also as a USB stick. Remove for this purpose the USB protective cap at the upper end of the device. The cap is connected by a short strap with the rest of the device, so that you cannot lose the cap. If you do not plug the **eVito collect** in the USB Slot, you should always leave the cap on the device.

III. Setup and operation functions

After inserting the battery, your **eVito collect** is ready for use. Restart the device by holding the button on the front for more than 4 seconds. **eVito collect** starts and shows you the battery status, as described previously. Next the device checks whether a SIM card has been inserted.

If no SIM card is inserted, the blue LED will first flash for 10 seconds. Then the green LED lights permanently. This indicates that the **eVito collect** is turned on and ready. The device can receive data via ANT; this is not transferred via GSM! For the mobile data transfer you will need a SIM card.

If the SIM card is inserted, **eVito collect** starts automatically with the GSM communication. If the communication is successful, the blue LED lights steadily in operation mode.

If the GSM communication is not possible, the blue LED flashes slow and constantly.

Device switch on/off

You can switch off the device at any time. Press the button on the front side during the operation for 4 seconds. All LEDs will flash briefly. After 2 seconds **eVito collect** turns off.

It is recommended to activate the lock switch (slide down from the cap), to prevent pressing the button accidentally.

Automatic data transmission

The **eVito collect** records measured values via integrated ANT module from other ANT-compatible instruments (blood pressure or blood glucose monitor, step counter...) on the internal SD-card. This data will be automatically transmitted by GSM (if SIM card is inserted).

eVito collect receives data from ANT devices and the green LED flashes. The data is stored on the SD card and for transmitting via GSM. During the transfer process, the green LED flashes.

The **eVito collect** will now transfer all stored data that have not been transmitted yet. During the transfer process, the blue LED flashes.

Locking your device

To prevent that during the operation the main button is pressed, you can lock it by key lock on the side of the device. Slide the switch to lock down, in the opposite direction of the USB protection cap.

Energy saving mode

To save power, **eVito collect** comes in the sleep mode after 30 seconds out of use. Then all the LEDs dim. If there are no operations all LED go out. When data is transferred, the LEDs lights dim. If you want to use the **eVito collect** again, then press 1 time briefly on the main control button. Then you will see the battery status.

Declaration of conformity

Products with CE label comply the R&TTE Directive (99/5/EC), which were published by the European Commission.

WEEE- notices

The WEEE (Waste Electrical and Electronic Equipment) Directive that came into effect as European law on 13th February 2003, led to a major change in the disposal of electrical equipment.

The primarily purpose of this directive is to prevent of electronic waste (WEEE), while encouraging for reuse, recycling and other forms of reprocessing to reduce waste.

The WEEE logo on the product and the packaging indicates that this product must not be placed in municipal waste. You are responsible to dispose all electrical and electronic equipment to the designated collection points. A separate collection and proper recycling of electronic products helps to preserve natural resources. Furthermore, the recycling of electrical waste is a contribution to our environment and therefore also for the health of all people. For more information about recycling about electrical and electronic equipment, recovery, and the container parks, you receive from local authorities, Waste management companies, in stores and from the device manufacturer.

RoHS-compliance

This product complies with Directive 2002/95/EC of the European Parliament and the Council of 27th January 2003 regarding the limited use of dangerous substances in electrical and electronic equipment (RoHS) and its variations.

Technical specifications

- *Communication: GSM/GPRS, ANT, USB*
- *Storage: 1 GB Flash RAM*
- *Date and time memory*
- *Plug: USB*
- *Battery: rechargeable, 700 mAh*
- *Dimensions: 85 x 30 x 10 mm*
- *Weight: 20g*
- *LEDs: 3 (blue, green, red)*
- *Buttons: Main control knob, 1 Lock key*

Warranty

HMM Diagnostics products are subject to strict quality requirements. For this reason, HMM Diagnostics GmbH provides the **eVito collect** with 2 year warranty. Please store the warranty card included in your product set.

Wear parts, SD-card, protection cap etc. are not subject of warranty.

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